INVENTUM

maakt 't moment

THERMOMETER

thermometer

TMC609

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English

	normalized symbols	page 22
	introduction	page 23
1	safety instructions	page 23
2	appliance description	page 26
3	body temperature	page 28
4	before first use	page 28
5	how to measure forehead temperature	page 29
6	how to measure object temperature	page 30
7	temperature taking tips	page 30
8	fever indication	page 31
9	check the memory	page 31
10	replacing the batteries	page 32
11	specifications	page 33
12	cleaning & maintenance	page 34
13	troubleshooting	page 35
14	important information regarding electromagnetic compatibility (EMC)	page 36
	<i>General terms and conditions of service and warranty</i>	page 43

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normalized symbols

Applied part of type BF

Disposal in accordance with directive 2012/19/EU

Refer to Instruction Manual

The name and the address of the manufacturer

The name and the address of the authorised representative in the European Community

Complies with the European Medical Device Directive (93/42/EEC) and amended by directive 2007/47/EC requirements. Notified body TÜV Rheinland (CE0197)

Keep dry

Caution

IP22

The first number 2: Protected against solid foreign objects of 12.5 mm Ø and greater. The second number: Protected against vertically falling water drops when enclosure tilted up to 15°. Vertically falling drops shall have no harmful effects when the enclosure is tilted at any angle up to 15°, on either side of the vertical.

Exhausted batteries do not belong in household waste. Dispose of the batteries according to the current federal, state and local regulation. As a consumer, you are obligated by law to return spent battery.

NOTES:

1. Under the environment with electrostatic discharge, the unit may malfunction and may require user to reset the unit.
2. Dispose with care; please consult the retailer for details.

introduction

The thermometer has been carefully developed for accurate, safe, fast and contact-free human body temperature measurements (at a distance of approximately 1-5cm) on the forehead and object temperature measurements.

The reference body site is core and the mode of operation is adjusted mode. The device is intended for the intermittent monitoring of human body temperature in the home.


It is intended for use on people of all ages. The patient can measure and change batteries under normal circumstances and maintain the device and its accessories according to the user manual.

1 safety instructions

However, as with any thermometer, proper technique is critical to obtaining accurate temperatures.

Therefore, in order to obtain an accurate measurement, please read this manual carefully before using.

- Use of this thermometer is not intended as a substitution for consultation with your physician. Measurement results are for reference only. Contact your physician if you have or suspect any health concerns.
- The infrared sensor must be kept clean, dry, and undamaged at all times to ensure accurate measurements.
- Lipid on the forehead may cause an inaccurate measurement, so make sure the forehead is clean.
- Don't touch or blow on the infrared sensor. A polluted infrared sensor may cause inaccuracy.
- Do not use this thermometer for measurements other than human body temperature, room temperature or object temperature.
- The temperature of the storage area differs a lot from that of the measuring area, please wait for the thermometer temperature to equalize to the room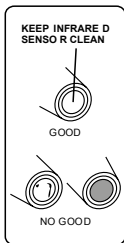

temperature about 30 minutes before use.


- Keep the device out of the reach of children and pets to avoid inhalation or swallowing of small parts.
- Do not allow children to take their temperatures unattended. Children may not be able to use the device according to the instructions in this user manual. It is not a toy.
- Don't throw battery into fire. Battery is likely to explode in fire.
- Don't store the thermometer to temperature extremes (below -20°C/-4°F or over 55°C/131°F) or humidity extremes (below 15%RH or over 90%RH). Failing to do so may cause inaccuracy.
- Don't use the thermometer if the main body is damaged (for example, the infrared sensor is broken). The continuous use of a damaged unit may cause injury, improper results, or serious danger.
- Don't take apart, repair or change any parts of the unit at any time.
- Don't use a mobile phone nearby when the thermometer is working.
- If your thermometer is not going to be used for a long time, it is advised to, remove the battery to prevent possible damage due to chemical leakage. If the battery does leak, remove carefully. Do not allow bare skin to touch leaking fluid.
- This product needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided, and this unit can be affected by portable and mobile RF communications equipment.
- The materials (e.g. ABS) of expect contact with patient had passed the ISO 10993-5 and ISO 10993-10 standards test, no toxicity, allergy and irritation reaction. However, based on the current science and technology, other potential allergic reactions are unknown.
- If you have an allergic reaction to materials, please stop treatment immediately and consult your physician.
- Store the thermometer in a dry, clean place. It is forbidden to leave the product exposed to any chemical solvent, lint, dust, direct sunshine or high temperature.
- When using, you shall not touch the batteries and the patient simultaneously.
- Do not maintain or service the device while the device is in use.

If you have any problems with this device, such as setting up, maintaining or using, please contact your customer department of the manufacturer.


Caution

This device should not be used adjacent to or stacked with other equipment and that if adjacent or stacked use is necessary, this device should be observed to verify normal operation in the configuration in which it will be used.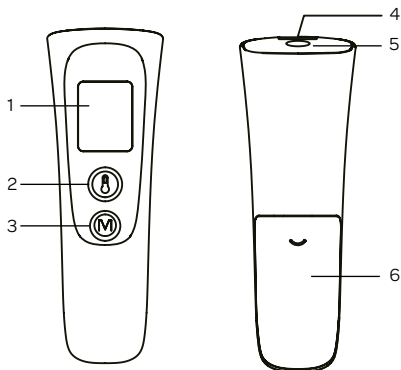

1. LCD display
2. Measuring button - 
3. Memory button - **M**
4. Distance sensor
5. Infrared sensor
6. Battery compartment

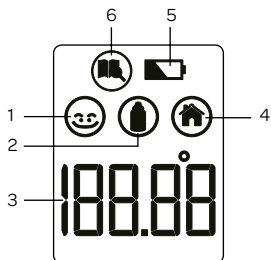1. Forehead temperature symbol
2. Object temperature symbol
3. Temperature display
4. Ambient temperature symbol
5. Low battery symbol
6. Memory symbol

3 body temperature

The normal body temperature is a range. A person's normal temperature range tends to decrease with increasing age. The following table shows normal temperature ranges by age:

0 ~ 2 years	36.4 ~ 38.0 °C	97.5 ~ 100.4 °F
3 ~ 10 years	36.1 ~ 37.8 °C	97.0 ~ 100.0 °F
11 ~ 65 years	35.9 ~ 37.6 °C	96.6 ~ 99.7 °F
> 65 years	35.8 ~ 37.5 °C	96.4 ~ 99.5 °F

The range of normal temperature varies from person to person and can be influenced by many factors such as time of day, level of activity, medications, emotion and so on.

So we recommend that you practice with the thermometer on yourself and family members when you are healthy. This way you know how the thermometer works and you feel more confident of the measurements you take when a family member is ill.

4 before first use

Before you use the appliance for the first time, please do as follows: carefully unpack the appliance and remove all the packaging material. Keep the material (plastic bags and cardboard) out of reach of children. Check the appliance after unpacking for any damage which may have occurred during transport.

1. To achieve accurate measurements, it is very important to check that the infrared sensor is clean before using. To clean the sensor, gently wipe its surface with a cotton swab that is slightly moistened with alcohol and immediately wipe dry with a clean cotton swab. After cleaning, allow at least 20 minutes drying time before taking temperatures.
2. Lipid on the forehead may cause an inaccurate measurement, so make sure the forehead is clean. If you clean the forehead, please wait for 5~10 minutes before measuring.

3. If the temperature of the storage area of the thermometer differs a lot from that of the measuring area, please wait 30 minutes before measuring.
4. Please open the battery cover and remove the insulating piece for the batteries when you use the thermometer for the first time. The thermometer turns on automatically and the LCD display looks like the figure as shown on the right.
Press any button to turn it on when it was turned off.

Caution:

In waiting mode, the LCD circularly displays the time and the current room temperature.

5 how to measure forehead temperature

1. Place the thermometer in your hand as shown with your thumb on the [8] measuring button.
 - Do not press the button yet.
 - Make sure the probe is clean before measuring.
2. Press the [8] measuring button. The sensor is on and you can locate the position for the forehead measurement. If the position is OK, release the button, but keep the thermometer in position until you hear a sound signal. The control light will be lit according to the temperature (green is good, red means fever). The measurement is complete and the thermometer may be removed from the head.
3. Read the temperature on the display.

Caution: After pressing the measuring button, the distance sensor will detect the distance between the forehead and thermometer. If the distance is between 1- 5cm, it will directly test the forehead temperature. If the distance is more than 5cm or less than 1cm, a sound signal will be issued, it indicates that the thermometer should be moved closer to the forehead in 1- 5cm. If not, it will return directly to waiting mode.

After the test, press the **[M]** memory button or wait for 1 minute to return to the waiting mode.

6 how to measure object temperature

1. Press at the same time on the buttons **[8]** and **[M]** to switch to the object mode.

2. Hold the thermometer and ensure the sensor is around 1 to 5 cm from the liquid or surface you want to measure. Press the **[8]** measuring button to start the measurement.

3. Remove the thermometer from the object. The LCD displays the measured temperature. You can press the **[M]** memory button to return back to standby mode. The thermometer will automatically return back to standby mode after 1 minute if no button is pressed.

7 temperature taking tips

1. External factors may influence forehead temperatures, particularly when an individual has:
 - been exposed to very hot or very cold temperatures
 - been recently swimming or bathing.
 - had their forehead covered.

In these cases, remove the individual from the situation and wait for 30 minutes prior to taking a temperature again.

2. With object measurement, the current surface temperature of the object is displayed. It can be different from its internal

temperature, especially if the surface is exposed to direct sunlight or a draught.


8 fever indication

There are two colors of the LED temperature indicator in this device. The relevant LED can show one of two lights after the measurement.

The detail range refers to following:

Green LED: 34.0°C ~ 37.9°C (93.2°F ~ 100.3°F)

Red LED: 38.0°C ~ 42.9°C (100.4°F ~ 109.3°F)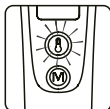

9 check the memory

There are a total of 9 memory positions for recording forehead temperature measurements.

The current measurement is always stored in the last storage space. When all storage spaces are occupied, the oldest measurement is always deleted from the memory.



Press the **[M]** memory button to check the memory data.

The LCD displays memory position number and measured temperature like the figures shown on the right.

Press the **[M]** memory button repeatedly to go to the memory position you need.

Press the **[8]** and **[M]** buttons at the same time, to go back to standby mode.

Attention: It is not possible to delete an individual measurement or all measurements manually.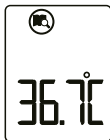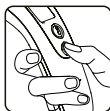


10 replacing the battery

When the low battery indicator “” appears on the LCD, or the thermometer does not function at all, you should replace the batteries as soon as possible.

The thermometer is supplied with two batteries (2 x AAA). Remove the battery cover and take out the batteries. Insert new AAA batteries into the battery compartment.

Make sure you are installing the batteries properly, be sure to match the positive and negative ends of the battery to the marking in the battery compartment of the thermometer.

Slide the battery compartment cover back until it snaps in place.

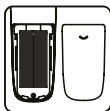Caution:

- To protect the environment, dispose of exhausted batteries according to national or local regulations.
- Keep batteries out of the reach of children.

11 specifications

Power supply	DC 3V (2 x AAA)
Body measuring range	32.0°C ~ 42.9°C (89.6°F ~ 109.3 °F)
Forehead measuring accuracy	± 0.2°C (± 0.4°F) from 35.0°C ~ 42.0°C (95.0°F ~ 107.6°F) ± 0.3°C (± 0.5°F) for other range
Object measuring range	-22°C ~ 80.0°C (-7.6°F ~ 176.0°F)
Object measuring accuracy	± 2.0°C (± 4.0°F)
Displayed room temperature range	5.0°C ~ 59.9°C (41.0°F ~ 139.8°F)
Room temperature accuracy	± 1°C (± 2°F)
Measuring time	1 second
Measuring interval	1 second
Dimensions	130.1 mm x 45.0 mm x 50.3 mm (LxWxD)
Weight	72 g (without batteries)
Forehead and object operating condition	15.0°C ~ 40.0°C (59.0°F ~ 104.0°F) with a relative humidity of 15% ~ 85% Atmosphere pressure: 700hPa~1060hPa
Storage and transportation condition	-20°C ~ 55°C (-4°F ~ 131°F) with a relative humidity of 15% ~ 90% Atmosphere pressure: 700hPa~1060hPa

Service life of the battery: with a new battery (carbon) approx. 6 months based on the use frequency of 5 times per day.

Caution:

This thermometer has been thoroughly tested and inspected to assure proper performance and operation!

12 cleaning & maintenance

To ensure accurate measurements, keep the infrared sensor and the distance sensor clean and free of scratches. This is very important. Finger prints or dirt will affect the accuracy of the thermometer.

In order to get an accurate measurement, please clean the infrared sensor and the distance sensor. Gently wipe its surface with a cotton swab slightly moistened with alcohol and immediately wipe dry with a clean cotton swab.

Use a soft dry cloth to clean the body of the thermometer. Never clean the thermometer with an abrasive cleanser, thinner, benzene or submerge the thermometer into water or other liquids.


After cleaning, store it in a clean, dry place at room temperature. Never expose the thermometer to extreme temperatures, humidity, direct sunlight or shock.

CALIBRATION

The thermometer is initially calibrated at the time of the manufacture. If the thermometer is used according to the instruction, periodic recalibration is not required.

Don't attempt to modify or reassemble the thermometer.

13 troubleshooting

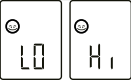Troubles	Checklist	Countermeasures
No response/ Automatic reset when pulling out the insulator	Batteries used up?	Change the batteries
	Batteries in wrong polarity?	Take out the batteries, reinsert batteries correctly
	Poor battery contact	
	The measured temperature is lower than 32.0°C/89.6°F or higher than 42.9°C/109.3°F. Please check the operation method.	Follow the steps in the user manual for proper measurement.
	Hardware problem	Contact the manufacturer or point of sale.
	The operating temperature is out of the range	Use the thermometer in the range of the operating conditions
The measurement is not accurate or if there is any doubt of the measure result	Please check if the infrared sensor is clean or not	Clean the infrared sensor with a cotton swab according to the user manual.
	Please check that the measuring way is correct or not	Ensure you have read the manual and know how to use the thermometer properly.
	Please check if you have let the thermometer and patient get stabilized in the room for 30 minutes at least	Please keep the thermometer and patient in the measuring room at least 30 minutes before using.
	Are you using the thermometer indoor?	Please take the measurement indoor.

The measurement is not accurate or if there is any doubt of the measure result	Please check if you have hold the thermometer in your hand too long and affect the accuracy	Put the thermometer on the table in the room where the measurement is taking place and let it cool down first.
	Please check if there is low battery icon on the LCD display	Replace with new batteries

14

Important information regarding electromagnetic compatibility (EMC)

MEDICAL ELECTRICAL EQUIPMENT needs special precautions regarding EMC. Portable and mobile RF communications equipment can affect MEDICAL ELECTRICAL EQUIPMENT.

Non-life support equipment separation distance calculation (3Vrms/3V/m compliance)			
Rated maximum output power of transmitter (Watt)	Separation distance according to frequency of transmitter (meter)		
	150kHz to 80Mhz $d = 1.2 \sqrt{P}$	80Mhz to 800Mhz $d = 1.2 \sqrt{P}$	800Mhz to 2.5Ghz $d = 1.2 \sqrt{P}$
0.01	0.12	0.12	0.23
0.1	0.37	0.37	0.74
1	1.17	1.17	2.33
10	3.69	3.69	7.38
100	11.67	11.67	23.33

Guidance and manufacturer's declaration - electromagnetic immunity			
The device is intended for use in the electromagnetic environment specified below. The customer or the user of these Clinical IR thermometer should assure that it is used in such an environment.			
Immunity test	IEC 60601 Test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact ±8 kV air	Complies	Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30%.
Radiated RF IEC 61000-4-3	3V/m 80 MHz to 2.5 GHz	Complies	Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, should be less than 3V/m. Interference may occur in the vicinity of equipment marked with the following symbol:
Conducted RF IEC 61000-4-6	Not applicable	Not applicable	 Separation distance calculation provided below. If a known transmitter is present, the specific distance can be calculated using the equations.

Immunity test	IEC 60601 Test level	Compliance level	Electromagnetic environment - guidance
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines ±1 kV for input/output lines	Not applicable	The ME equipment is solely battery powered.
Surge IEC 61000-4-5	±1 kV differential mode ±2 kV common mode	Not applicable	
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3A/m	Complies	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	> 95% dip 0.5 cycle 60% dip 5 cycles 70% dip 25 cycles 95 dip 5 sec.	Not applicable	The ME equipment is solely battery powered.

Guidance and manufacturer's declaration - electromagnetic emissions

The device is intended for use in the electromagnetic environment specified below. The customer or the user of these Clinical IR thermometer should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	Complies
Harmonic emissions IEC 61000-3-2	Not applicable	The ME equipment is solely battery powered.
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Not applicable	

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We do not need to remind you of the importance of service. After all, we develop our products to a standard so that you can enjoy them for many years, without any concerns. If, nevertheless, there is a problem, we believe you are entitled to a solution straight away. Hence our products come with an exchange service, on top of the rights and claims you are entitled to by law. By exchanging a product or part, we save you time, effort and costs.

2-year full manufacturer's warranty

1. Customers enjoy a 2-year full manufacturer's warranty on all Inventum products. Within this period, a faulty product or part will always be exchanged for a new model, free of charge. In order to claim under the 2-year full manufacturer's warranty, you can either return the product to the shop you bought it from or contact the Inventum customer service department via the form at www.inventum.eu/service-aanvraag.
2. The 2-year warranty period starts from the date the product is bought.
3. In order to claim under the warranty, you must produce a copy of the original receipt.
4. The warranty applies only to normal domestic use of Inventum products within the Netherlands.

Breakdowns or faults outside the warranty period

1. Breakdowns or faults in small or large domestic appliances outside the warranty period, can be reported to the customer services department via the form at www.inventum.eu/service-aanvraag or by calling the customer services department.
2. The customer services department may ask you to send the product for inspection or repair. The costs of dispatch will be at your expense.
3. The inspection to establish whether repair is possible involves a charge. You need to grant your permission for this, in advance.
4. In the event of a large domestic appliance, Inventum, at your request, can send out a service engineer. In that case, you will be

charged the call-out costs, as well as parts and labour.

5. In the event of an instruction to repair, the repair costs must be paid in advance. In the event of a repair by a service engineer, the costs of the repair must be settled with the engineer onsite, preferably by means of PIN payment.

Warranty exclusions

1. The following is excluded from the aforesaid warranties:
 - normal wear and tear;
 - improper use or misuse;
 - insufficient maintenance;
 - failure to comply with the operating and maintenance instructions;
 - unprofessional installation or repairs by third parties or the customer himself;
 - non-original parts used by the customer;
 - use for commercial or business purposes;
 - removal of the serial number and/or rating label.
2. In addition, the warranty does not apply to normal consumer goods, such as:
 - dough hooks, baking tins, (carbon) filters, etc.;
 - batteries, bulbs, carbon filters, fat filters etc.;
 - external connection cables;
 - glass accessories and glass parts such as oven doors;
 - and similar items.
3. Transport damage not caused by Inventum is also excluded. Therefore, inspect your new device before starting to use it. If you detect any damage, you must report this to the store where you purchased the product within 5 working days, or to the Inventum customer service department via the form at www.inventum.eu/service-aanvraag. If transport damage is not reported within this period, Inventum does not accept any liability in this respect.
4. The following are excluded from warranty and/or replacement: faults, loss of and damage to the device as a result of an event that is normally insured under the home contents insurance.

Important to know

1. The replacement or repair of a faulty product, or a part thereof, does not lead to an extension of the original warranty period.
2. If a complaint is unfounded, all costs arising from it will be at the customer's expense.
3. Following expiry of the warranty period, all costs of repair or replacement, including administration costs, dispatch and call-out charges, will be charged to the customer.
4. Inventum cannot be held liable for damage caused by external events, unless this liability arises from mandatory statutory provisions.
5. These warranty and service provisions are governed by Dutch law. Disputes will be settled exclusively by the competent Dutch court.

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Shenzhen Dongdixin Technology Co., LTD.
Inventum Huishoudelijke Apparaten B.V.
Shanghai International Holding Corp. GmbH (Europe)